




## NEW Environmental Swab Collection Kit for Anthrax

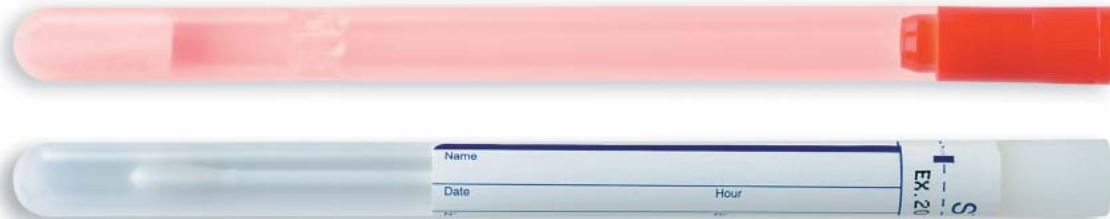

### Each Sterile Peel Pack:

Red color coded tube containing sponge soaked with hydrating solution\*

White color coded tube containing plain dry swab

\* Hydrating solution can be PBS or PBS plus Tween 80 or any custom formulation!

### Simple, Ready to Use Kit

- ▶▶ Color coded components so the product can be used by inexperienced or non-scientific field workers
- ▶▶ Shelf life 12 - 15 months at room temperature
- ▶▶ Custom formulations available
- ▶▶ Dry swab tube can be processed directly in the lab. Medium can be added to the dry swab tube and sample can be heat shocked in a water bath
- ▶▶ Low cost solution





# How to Use the New Swab Collection Kit for Anthrax



Peel open pack exposing the tube caps



Remove red color coded cap




Remove white cap swab



Place swab in red tube



Close cap so swab is in contact with sponge in red tube

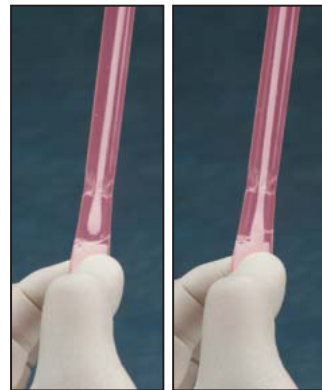

Squeeze base of red tube to moisten swab tip



Remove swab from red tube




Sample the surface or equipment



Return swab to white tube



Remove white tube from package

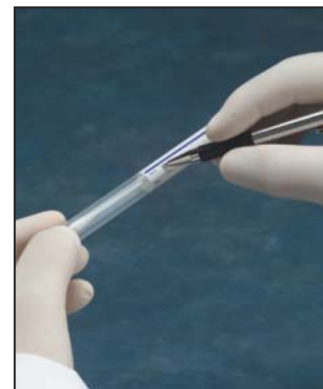

Write sampling site information on white tube label and ship sample to lab

