



Back to Work Sample Collection for COVID-19 Surveillance for Pooled Samples






Screen Employees and Students for COVID-19 Using a Cost-Saving Pooled Testing Method

SAMPLE POOLING is a cost-effective method for screening groups where the expected positivity rate is low. PodSwab™ is a sample collection kit with one tube of UTM® Universal Transport Media™ paired with five flocked swabs for testing groups or pods of people, helping businesses and schools efficiently screen employees and students, saving costs and time.



How Does Sample Pooling Testing Work?

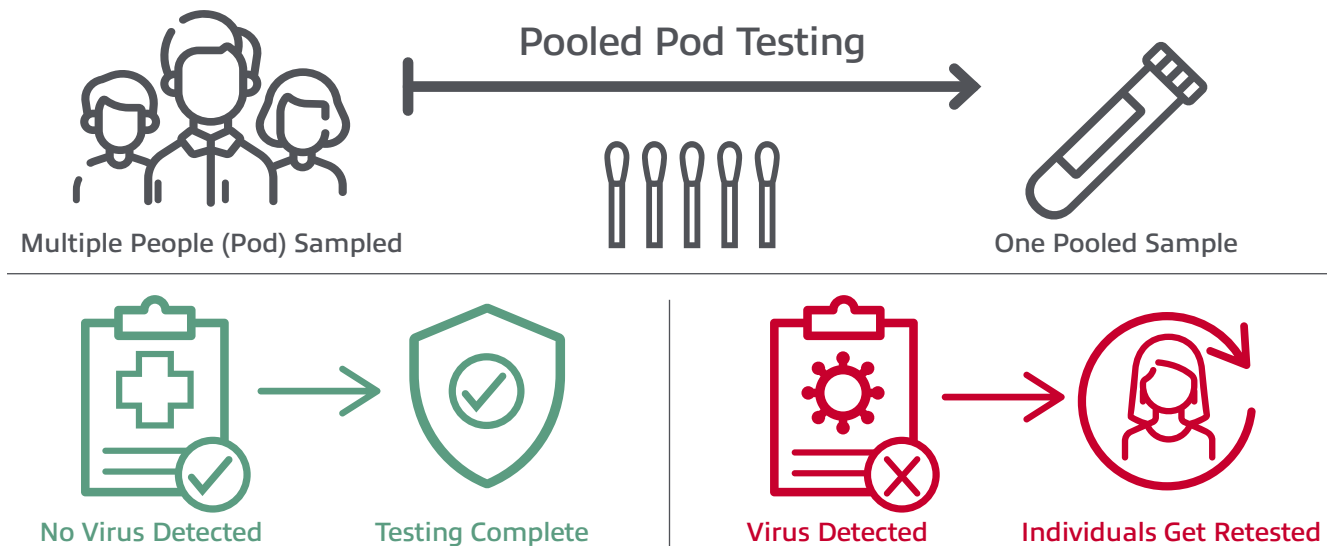

Designed to **streamline** and ease the burden of COVID-19 testing, PodSwab™ consists of **6 mL UTM® Universal Transport Media** and **5 nasal flocked swabs** allowing for up to five different individual samples in each tube.

UTM™ is an FDA-cleared collection and transport system suitable for **collection, transport, maintenance of clinical specimens containing viruses, including COVID-19**. The system maintains organism viability for 48 hours at room or refrigerated temperature.




Collection, Transport and Preservation of Pooled Specimens for COVID-19 Screening

SAMPLE COLLECTION should be performed in accordance with the test manufacturer’s package insert. These illustrations serve as a general guideline for sample collection.



COLLECT

Collect the sample using the swab. Avoid touching the swab below the breakpoint.

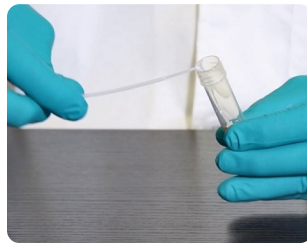

SNAP

Remove the cap. With the tube pointed away from the face insert the swab so the breakpoint lines up with the edge of the tube, and bend to break.



REPEAT

Screw the cap on tightly and set aside. Repeat steps 1 and 2 with the rest of the pod until up to 5 individual samples have been collected.



SEND

Identify tube with information from each person sampled and follow testing laboratory instructions for shipping tubes for testing.

PART NUMBER	PRODUCT DESCRIPTION	UNIT OF MEASURE
3C062N	Each PodSwab™ Consists of: One tube containing 6 mL UTM® Medium and 5 individually wrapped flocked swabs	10 PodSwabs™ per box (10 tubes and 10 packs of 5 swabs)

Interested in seeing if PodSwab™ is the right choice for your Pod?



Request Samples



Order Now!



Ask a Question

(800) 216-4016 | copanusa.com