



Let's Get Back to Work.

Save on back to work COVID-19 screening by purchasing sample collection supplies directly from the leading manufacturer in sample collection!





Does your organization screen work groups for COVID-19, or are you considering implementing a screening program?



Did you know you can substantially reduce costs for this type of testing using sample pooling?

SAMPLE POOLING is a cost effective method for screening groups where the expected positivity rate is low. PodSwab™ is a sample collection kit with one tube of UTM® Universal Transport Media™ paired with five flocked swabs for testing groups or pods of people, helping businesses and schools efficiently screen employees and students, saving costs and time.



How Does Pod Testing Work?



Multiple People (Pod) Sampled

Pooled Pod Testing




One Pooled Sample



No Virus Detected



Testing Complete

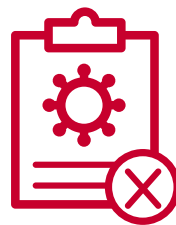

Virus Detected



Individuals Get Retested




Designed to **streamline** and ease the burden of COVID-19 testing, PodSwab™ consists of **6 mL UTM®** and **5 nasal flocked swabs** allowing for up to five different individual samples in each tube.

Universal Transport Media or UTM® is an FDA-cleared collection and transport system suitable for **collection, transport, maintenance, and long-term freeze storage of clinical specimens containing viruses, including COVID-19.** The transport medium-filled tube, complete with plastic screw cap, maintains organism viability for 48 hours at room or refrigerated temperature.



COLLECT

Collect the sample using the swab. Avoid touching the swab below the breakpoint.

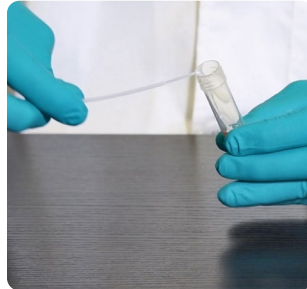

SNAP

Remove the cap. With the tube pointed away from the face insert the swab so the breakpoint lines up with the edge of the tube, and bend to break.



REPEAT

Screw the cap on tightly and set aside. Repeat steps 1 and 2 with the rest of the pod until up to 5 individual samples have been collected.



SEND

Identify tube with information from each person sampled and follow testing laboratory instructions for shipping tubes for testing.



Interested?



CONTACT your testing laboratory to let them know you'd like to set up a back to work screening program and ask them about sample pooling using sample pods



INFORM the provider that you will obtain nasal sample collection kits to pool 5 samples into one tube for testing.



THE LABORATORY will work with you to set up a screening program for your employees, sending labels and sample collection instructions.



COLLECT samples and return to testing laboratory per their instructions.

PART NUMBER	PRODUCT DESCRIPTION	UNIT OF MEASURE
3C062N	Each PodSwab™ Consists of: One tube containing 6 mL UTM® Medium and 5 individually wrapped flocked swabs	10 PodSwabs™ per box (10 tubes and 10 packs of 5 swabs)

Interested in seeing if PodSwab™ is the right choice for your Pod?



Request Samples



Order Now!



Ask a Question

(800) 216-4016 | copanusa.com