

 **UTM**[®] UNIVERSAL TRANSPORT MEDIUM

COLLECTION, TRANSPORT AND PRESERVATION OF CLINICAL SPECIMENS FOR VIRAL MOLECULAR DIAGNOSTIC TESTING



ROOM TEMPERATURE
STABLE



CLEARED FOR
USE WITH VIRAL
MOLECULAR ASSAYS




MAINTAINS VIABILITY
OF ORGANISMS FOR
UP TO 48 HOURS




CONTAINS
ANTIBIOTICS TO
INHIBIT BACTERIAL
AND FUNGAL FLORA
IN PATIENT SAMPLES

UTM[®] Collection Quick Guide

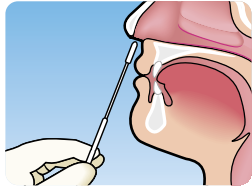

COLLECT

Collect the patient sample using the swab. To limit the risk of contamination, be sure that the swab tip only touches the collection site and avoid touching the swab below the breakpoint.

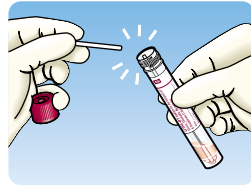


SNAP

Remove cap from the tube and insert the swab to the bottom of the tube.

Holding the tube away from face, grasp the end of swab shaft and bend at a 180-degree angle to break at the breakpoint. If needed, gently rotate the swab shaft to complete the breakage.

Screw the cap on tightly to prevent leakage.

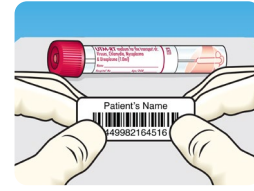

SEND

Identify tube with patient information and send to laboratory.

Specimen collection should be performed by health care personnel who have completed training and demonstrated competency. Always read the manufacturer's package insert for specific instruction regarding specimen collection and transport for the type of test kit being used.

Product Description

DESCRIPTION	CAT. NO.	PACKAGING
MEDIA ONLY		
3 mL UTM[®] Medium in 16x100mm Tube	3C047N	50/Pack; 6 Packs/Case
KITS		
Minitip Size Flocked Swab with Breakpoint Packaged with 3 mL UTM [®] Medium in 16x100mm Skirted Tube with Plastic Red Cap	3C057N	50/Pack; 6 Packs/Case
Regular Size Flocked Swab with Breakpoint Packaged with 3 mL UTM [®] Medium in 16x100mm Skirted Tube with Plastic Red Cap	3C064N	50/Pack; 6 Packs/Case
Minitip Size Flocked Swab with Breakpoint Packaged with 3 mL UTM [®] Medium in 12x80mm Tube with Plastic Red Cap	3C071N	100/Case
Regular Size Polyester Swab with Breakpoint Packaged with 3 mL UTM [®] Medium in 16x100mm Skirted Tube with Plastic Red Cap	3C059N	50/Pack; 6 Packs/Case
Minitip Size Foam Swab with Breakpoint Packaged with 3 mL UTM [®] Medium in 16x100mm Skirted Tube with Plastic Red Cap	3C058N	50/Pack; 6 Packs/Case

Individual swabs sold separately, contact COPAN for more details.



COPANUSA.com
 800.216.4016
 @COPANUSA

info@copanusa.net
 facebook.com/COPAN.USA

Made in the USA