

FLOQSwabs®

**Better Specimen Collection
Better Diagnostics**



Copan
innovating together



COPAN FLOQSwabs® consist of molded plastic applicator shafts with tips (in varying dimensions) coated with Nylon® fibers, creating a thin absorbent layer allowing for quick sample uptake and elution of more than 90% of the sample.¹ The unique swab design ensures the most effective specimen collection to expand downstream testing capabilities.²



Superior sample collection

FLOQSwabs® have no internal core to trap the sample, allowing rapid absorption and ensuring fast and complete specimen elution.



Designed for multiple collection sites

Various shaft and tip dimensions make FLOQSwabs® an improved alternative to invasive and costly collection procedures.^{4,5}



The perfect partner

FLOQSwabs® are compatible with a variety of bacteriology and virology transport media, ensuring broad assay compatibility.^{2, 3, 6-10}




Why Flocked?

Maximizing sample recovery allows multiple tests to be run from a single collection, and improves the sample yield from each collection, improving efficiency and diagnostic accuracy.

Traditional Fiber Swab

Yards of fiber wrapped around applicator

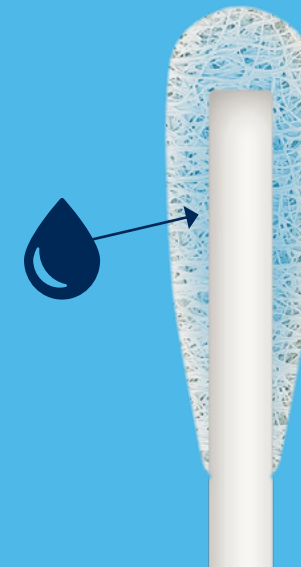


Sample is trapped in swab fibers

One swab for one test

FLOQSwabs®

Nylon® fibers are applied to applicator using a proprietary flocking process

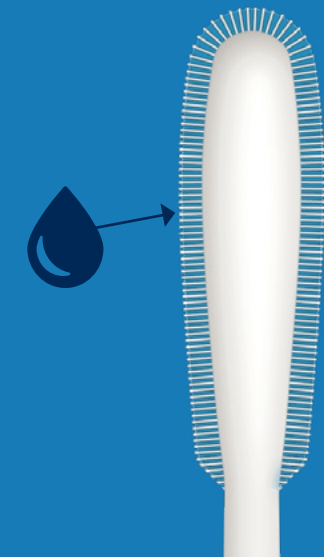

Sample diffuses and becomes trapped in the fibers.

Sample stays close to the surface, eluting upon contact with transport media. Brush like texture dislodges and collects cells.

More of sample available for testing

How many tests from one sample?

Multiple aliquots for various tests⁶⁻¹¹

Patent Information

US 8,317,728 8,114,027 8,979,784 9,011,358 13/657,949 16/448,398 9,173,779 14/448398 10,327,741 | EP1608268 | JP4579902 | AU2004226798 | CA2,515,205 | NZ541560 | SG170654 | CNZL200610099310.9

Ordering Information

FLOQSwabs® are available with various tip sizes and breakpoints to be used for various applications.

All FLOQSwabs® are packaged sterile and come in boxes of 100 pieces, 10 boxes to a case, unless otherwise noted.

[Learn More](#)

FLOQSwabs® in Peel Pouch

| Cat. No. | Product | Break Point | Commonly Used For |
|----------|---------|-------------|---|
| 519CS01 | REGULAR | 100 mm | Nasal, throat, vaginal, groin, axilla, rectal, wound, buccal, stool |
| 502CS01 | REGULAR | 80 mm | Nasal, throat, vaginal, groin, axilla, rectal, wound, buccal, stool |
| 520CS01 | REGULAR | 30 mm | Nasal, throat, groin, axilla, wound, buccal, stool |
| 518CS01 | MINITIP | 100 mm | Urethral, nasal, pediatric |
| 501CS01 | MINITIP | 80 mm | Urethral, nasal, pediatric |

FLOQSwabs® in Peel Pouch

| Cat. No. | Product | Break Point | Commonly Used For |
|-----------|------------------------------|-------------|-------------------|
| 525CS01 | ULTRA-THIN MINITIP | 80 mm | Urethral |
| 503CS01 | FLEXIBLE | 100 mm | Nasopharyngeal |
| 534CS01 | FLEXIBLE | N/A | Nasopharyngeal |
| 516CS01 | MINI FLEXIBLE | 100 mm | Neonatal |
| 56380CS01 | LONG CONTOURED WITH STOPPER | 80 mm | Mid Turbinate |
| 56780CS01 | CONTOURED WITH LOW STOPPER | 80 mm | Mid Turbinate |
| 56750CS01 | SHORT CONTOURED WITH STOPPER | 50 mm | Mid Turbinate |

FLOQSwabs® in Tube

| Cat. No. | Product | Break Point | Commonly Used For |
|----------|--------------------------|-------------|---|
| 519C | REGULAR | 100 mm | Nasal, throat, vaginal, groin, axilla, rectal, wound, buccal, stool |
| 552C | REGULAR | 80 mm | Nasal, throat, vaginal, groin, axilla, rectal, wound, buccal, stool |
| 520C | REGULAR | 30 mm | Nasal, throat, groin, axilla, buccal, stool |
| 5U002S | REGULAR WITH LOW STOPPER | 80 mm | Rectal, entrics |
| 528C | LARGE TIP WITH STOPPER | N/A | Buccal |
| 518C | MINITIP | 100 mm | Urethral, nasal, pediatric |
| 551C | MINITIP | 80 mm | Urethral, nasal, pediatric |

FLOQSwabs® in Tube

| Cat. No. | Product | Break Point | Commonly Used For |
|----------|--------------------|-------------|-------------------|
| 553C | FLEXIBLE MINITIP | 100 mm | Nasopharyngeal |
| 516C | MINI FLEXIBLE | 100 mm | Neonatal |
| 560C | ULTRA-THIN MINITIP | 80 mm | Urethral |

Scientific References

- Copan Group. Data on file - Internal elution studies.
- Munywoki, P. K., Hamid, F., et al. (2011). Improved detection of respiratory viruses in pediatric outpatients with acute respiratory illness by real-time PCR using nasopharyngeal flocked swabs. *Journal of Clinical Microbiology*, 49(9), 3365–3367. <https://doi.org/10.1128/JCM.02231-10>
- Copan Italia S.p.A. (2020). FLOQSwabs® in pouch: Product insert & how to use guide (HPC217 Rev.00). Copan Italia S.p.A.
- Saville, M., Hawkes, D., et al. (2020). *Analytical performance of HPV assays on vaginal self-collected vs practitioner-collected cervical samples: The SCoPE study*. *Journal of Clinical Virology*, 127, 104375. <https://doi.org/10.1016/j.jcv.2020.104375>
- Misra, A., Speicher, D. J., et al. (2021). *Self-collected oral flocked swabs to measure prevalence of Epstein-Barr Virus antibodies and DNA amongst university students*. *Diagnostic Microbiology and Infectious Disease*, 100, 115295. <https://doi.org/10.1016/j.diagmicrobio.2020.115295>
- Van, T. T., Cox, L. M., et al. (2017). *Prevalence of Fusobacterium necrophorum in children presenting with pharyngitis*. *Journal of Clinical Microbiology*, 55(4), 1147–1153. <https://doi.org/10.1128/JCM.02174-16>
- Cleveland Clinic Laboratories. (2024). *Microbiology specimen collection & transport guidelines*. Cleveland Clinic. Retrieved from <https://clevelandcliniclabs.com/laboratory-resources/specimen-collection-transport/>
- Fontana, C., Favaro, M., & Favalli, C. (2013). *How liquid-based microbiology can change the workflow in microbiology laboratories*. *Advances in Microbiology*, 3(6), 504-510. <https://doi.org/10.4236/aim.2013.36067>
- Plapp, F. (2014). E Swab Collection System. ClinLab Navigator. <https://www.clinlabnavigator.com/blog/e-swab-collection-system>
- Chernesky, M., Castriciano, S., et al. (2006). *Use of flocked swabs and a universal transport medium to enhance molecular detection of Chlamydia trachomatis and Neisseria gonorrhoeae*. *Journal of Clinical Microbiology*, 44(3), 1084–1086. <https://doi.org/10.1128/JCM.44.3.1084-1086.2006>
- Copan Diagnostics Inc. (2007). *ESwab: New liquid-based transport system for microbiology with flocked swabs*. Retrieved from Rapid Microbiology website: <https://www.rapidmicrobiology.com>



These products may not be available in all Countries. Please refer to www.copanusa.com for the most recent version of this document. Always consult product technical documentation for complete information. The use of this product in association with diagnostic kits or instrumentation should be validated by the end user. All trademarks, logos and brand names are the property of their respective owners. ©2025 COPAN Diagnostics Inc. All rights reserved.
Code: FLOQ Brochure_03_2025

 CopanUSA.com  **800.216.4016**  [@CopanUSA](https://twitter.com/CopanUSA)  communications@copanusa.net  [/Copan.USA](https://facebook.com/Copan.USA)

COPAN Diagnostics, Inc.

26055 Jefferson Avenue | Murrieta, CA 92562 USA